


ONE-CARE[®]

SAFETY PEN NEEDLES **DUO**

easydrip[™]



Discover our Innovative Duo Technology!

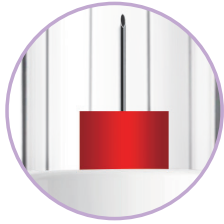

Injection needle automatically shielded before & after use



"Thin-Wall Technology" for optimal insulin flow

Ribbed main housing for convenient attachment to pen

Back-end needle automatically covered after removal from pen

SAFE

- Injection needle automatically shielded before and after use
- Back-end needle automatically covered after removal of the pen needle from the injection pen
- The needle front is pre-covered to reduce the risk of needle stick injuries
- Red color visual indication to easily distinguish between used and non-used device

CONVENIENT

- Less resistance during safety mechanism activation process and needle penetration

UNIVERSAL

- Compatible with most popular pens in the market
- Available in different needle configurations


WWW.ONE-CARE.COM




ONE-CARE[®]
SAFETY PEN NEEDLES **DUO**

INSTRUCTIONS FOR USE


- Easy-to-use design with intuitive handling and simple steps

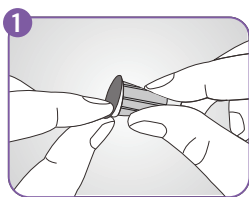


1 Peel off the sterility protective seal

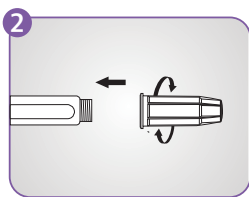


2 Screw the pen needle onto the pen in clockwise direction

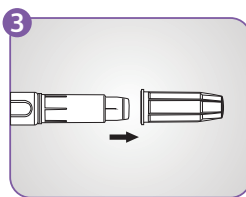


3 Remove the outer cover by pulling it straight off

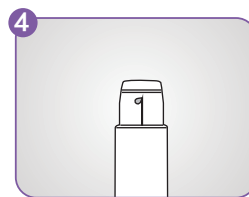

4 Set the dose on the pen and check the flow by priming the device

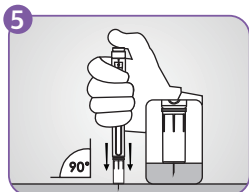


5 Insert the needle completely into the skin at a 90-degree angle and inject fluid

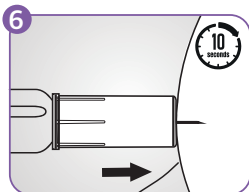

6 After fully depressing injection button of the pen injector, allow minimum 10 seconds for delivery of the entire dose before removing needle from the skin



7 Remove the pen needle from pen device by twisting it off

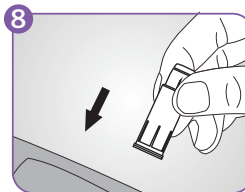

8 Dispose of the used pen needle into sharps container as per relevant regulations

COMPATIBILITY

Novo Nordisk	NovoPen® FlexPen®, NovoPen® FlexTouch®, NovoPen® Echo, NovoPen® 4, NovoPen® 5, InDuo®, InnoLet®, Victoza®
Eli Lilly	KwikPen®, Forteo®, Huminsulin®, HumaPen®, HumaPen® Savvio™, HumaPen® Luxura™ HD, HumaPen® MEMOIR™
Sanofi	Apidra® SoloSTAR®, ClickSTAR®, Lantus® SoloSTAR®, Toujeo® SoloSTAR®, AllSTAR™, JuniorSTAR®, Lyxumia®, OptiPen® Pro 1
BD	BD® Pen
Astra Zeneca	Byetta®
Owen Mumford	Autopen® Classic, Autopen® 24
B. Braun	Omnican® Pen 31
Ypsomed	Ypsopen®, OptiClik®
Berlin Chemie	BerliPen® Areo 2/3, BerliPen® Areo 301/302, BerliPen® Junior, BerliPen® Precision
Haselmeier	Diapen®, TactiPen®
Bioton	GensuPen®

*Brands are trademarks of their respective owners

ORDERING INFORMATION

- Available in different needle configurations :



REF	NEEDLE	COLOR	PACKAGING
3350	ONE-CARE® DUO Safety Pen Needles 30G x 5mm	■	100 pcs / bx 8 bx (800 pcs) / cs
3351	ONE-CARE® DUO Safety Pen Needles 30G x 8mm	■	100 pcs / bx 8 bx (800 pcs) / cs

WWW.ONE-CARE.COM

MediVena®
inspired by innovation

www.medivena.com

AMERICAS:
MediVena
1 Glenlake Parkway, Ste 700, Atlanta, GA 30328, USA
+1 404 514 2586 info@medivena.com
CUSTOMER SERVICE TOLL FREE: 888 270 6457

EUROPE:
MediVena Sp. z o.o.
Al. Jana Pawla II 27, 00-867 Warsaw, POLAND
+48 695 995 700

